



Lancing Device




VACCUM TECHNOLOGY

EASE THE PAIN

Soft Touch is a blood sampler that offers premium quality, extra value and vacuum technology.

Premium Quality Features

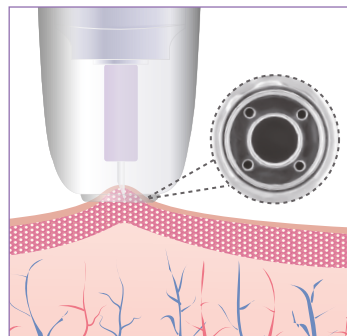

Almost painless

- **Vacuum technology.**
Multiple holes absorbs the skin firmly and draw it away from nerve endings to reduce pain dramatically.
- **Dual tracks for stable puncture depth.**
- **Precise penetration depth.**
Soft Touch Lancet have been specially designed with minimum variation to assure stable puncture depth, which greatly helps achieve painless blood sampling.



Adjustable

- **8 penetration depths meet multipel choices.**



Avoid Accidental Triggering

- **The button is designed on the top to avoid accidental triggering.**

One Step to Activate

- **Just push the top and release it to obtain a blood sample.**

Instruction for Use



1. Push the top button.



2. Remove the vacuum cap.



3. Rotate the dept adjuster to the appropriate position.



4. Put in a Soft Touch Lancet and screw off the cap.



5. Put on the vacuum cap.



6. Place the lancing device tightly on the blood collection area.



7. Push the top button and release it quickly.



8. Remove the lancing device slowly from sampling site.



9. Penetrate the lancet into its cap.



10. Throw the lancet into a sharps container.



11. Put back the vacuum cap.

COMPATIBLE LANCETS

Soft Touch lancing device is compatible with the most square shaft lancets. Soft Touch lancing device have been specially designed with minimum variation to assure stable puncture depth, which greatly helps achieve almost painless blood sampling. **Therefor, we recommend you to use Soft Touch lancing device with Soft Touch lancets to get better almost painless blood smapling experience.**

PUNCTURE DEPTH

Soft Touch lancing device has 8 gears, corresponding to 8 puncture depths, ranging from #1 (the shallowest) to #8 (the deepest). It is recommended to start the test from #1 according to the operating instructions for the first use. If no blood is produced, select the next gear until an appropriate amount of blood is obtained.

WARNINGS

Always dispone of the used lancet in a sharps container.

CAUTION

1. Soft Touch lancing device is for single patient use only. Never share the lancing device. Never reuse the lancets.
2. Do not leave the lancet in the lancing device after use.
3. A new lancet must be placed into the lancing device before use.

4. Be careful not to be pricked by the lancet whenever the lock cap is open with lancet loaded.
5. Expect following the use instructions, please don't disassemble the lancing device under any condition.
6. The lancing device contains some small parts which could result in a choking hazard for children.
7. The press force applied on test site may change the dept of puncture. To get almost painless blood sampling experience, keep a steady press force every time you use it.

CLEANING – After every use

We recommend wiping the lancing device, including the inside and outside of the adjusting cap and lock cap, with 70% isopropyl alcohol. After wiping, allow the lancing device to air dry completely.

STORAGE CONDITION

Avoid damp and direct sunlight.

USE LIFE

The period of validity is 5 years and it can not be used more than 3000 times.